

DRIFFIELD MOBILITY

Welcome To Your New Travel Scooter



Here are some handy care instructions to help you get the most out of your travel scooter.

DO



- Purchase a cover if you intend to use during rainfall.
- Store your scooter somewhere dry, either indoors, garage or shed.
- Turn the power off when transferring to and from the scooter.
- Avoid rough or soft terrain and long grass wherever possible.
- Turn the power off when not in use.
- Ensure your scooter battery is airline friendly if intended to be taken on an aeroplane
- Take a spare key with you when traveling just in case
- Remember to read the manufacturers instructions.



DON'T

- Leave your scooter outside in the rain or use during rainfall without a cover.
- Exit or enter the scooter unless the power is switched off.
- Forget to get your new scooter insured, especially if you regularly travel.
- Switch off the power whilst driving – this will cause the scooter to stop abruptly.
- Exceed the stated weight limit.
- Travel through water or over soft terrain (mud, sand etc.)
- Turn sharply at full speed.
- Leave on charge for more than 12 hours .

Thank you for your purchase